medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T + 32-11 24 25 60
 F +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Itapeva 538 cj. 42
 Bela Vista, São Paulo - SP
 CEP: 01332-000
 Brazil
 T +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Orient M.E.A. ZZC
 TAJURUS GROUP
 Suite No. M01 Wasel Al Mamzar Bldg.
 Al Wuhida Road, Al Mamzar
 P.O.BOX NO. 91464
 DUBAI, UAE
 United Arab Emirates
 T: +971-4 2557344
 F: +971- 4 2557399
 md@taurusgroup.net

medi Polska Sp. z o.o.
 Zygmunt Starego 26
 44-100 Gliwice
 Poland
 T +48-32 230 60 21
 F +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
 Unit 1806, Floor 18, Sandhill Plaza
 No. 1
 2290 Zuchongzhi Road
 200131 Shanghai
 People's Republic of China
 T: +86-1366 1668 687
 F: +86-2161 7696 07
 info@medi.cn

medi

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Important notes

This medical product is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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Polyamid, Polyester, Elasthan, Viskose

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Lumbamed[®] basic

Intended purpose

The Lumbamed basic is a lumbar support. The product is exclusively to be used for the orthotic fitting of the lower back and only on intact skin.

Areas of use

- Non-specific pain in the area of the lumbar spine
- Wear-related pain in the area of the lumbar spine
- Irritation of the sacro-iliac joint
- Excessive stress on the border between the lumbar spine and the pelvis
- Weak or imbalanced musculature in the trunk (back or stomach)
- Loosening of the pubic symphysis

Side-Effects / Contraindications

There are no known side effects if the correctly sized product is fitted as recommended. If unexpected pain, swelling or numbness occurs while wearing the support, remove it and seek medical advice. Particular care is required where the following conditions are present:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).

Any combination with other products must be discussed in advance with the responsible doctor.

Mode of Action/Comment on Wear

Please note that initial fitting should be supervised by a healthcare professional. With the combination of the compressive knitted fabric and the integrated back rods, Lumbamed basic

stabilises the area of the lumbar spine and the border to the pelvis, without impairing muscular strength. The static forces in the vertebral column are improved and stress on the affected segments is reduced. This alleviates pain.

Supports cause the greatest effect during physical exercise. In principal, the support can be worn all day. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and seek medical advice.

The support is also suitable for use in water. After use, please rinse out the product thoroughly.

Putting on / taking off

- The label on the right inside of the fastener explains how to put the support on (Fig. 1).
- First, slip into the wrist strap in the left inside of the fastener. Wrap the bandage around the body and then slip into the outer wrist strap on the right side (Fig. 2+3).
- Pull the two fastening sides over each other and then secure them as centrally as possible with the Velcro. The bandage has been correctly applied when the bottom edge of the fastener is located just above the pubic bone (Fig. 4+5).
- The small finger tab on the edge of the right side of the fastener will help you to undo the bandage again (Fig. 6).

The Clima Comfort knitted fabric will achieve its best breathing activity when the bandage is worn directly against the skin.

Care instructions

Velcro fasteners should be closed for washing. We recommend using a laundry bag. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- You can wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Polyamide, polyester, elastane, viscose

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

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