medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 Vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Orient M.E.A. ZZC
 TAURUS GROUP
 Suite No. M01 Wasel Al Mamzar Bldg.
 Al Wuhida Road, Al Mamzar
 P.O.BOX NO. 91464
 DUBAI, UAE
 United Arab Emirates
 T: +971-4 2557344
 F: +971- 4 2557399
 md@taurusgroup.net

medi Polska Sp. z o.o.
 Zygmunt Starego 26
 44-100 Gliwice
 Poland
 T +48-32 230 60 21
 F +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
 Rm 1806, Building 1 Sandhill Plaza
 No. 2290 Zuchongzhi Road
 Pudong New District
 200131 Shanghai
 People's Republic of China
 T: +86-21 61761988
 F: +86-21 61769607
 info@medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis
 Tic. AS
 Kültür mah. çaldıran sok.14/3
 06420 Çankaya/Ankara
 Turkey
 T +90 312 435 20 26
 F +90 312 434 22 82
 info@medi-turk.com
 www.medi-turk.com

medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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medi SAS[®] comfort

Intended Purpose

medi SAS comfort is a shoulder support cushion. The product is exclusively to be used for the orthotic fitting of the shoulder and only on unbroken skin or if any wounds have been properly covered.

Indications

- Postoperative immobilisation following the implantation of shoulder prostheses
- Immobilisation after the reconstruction of rotator cuff ruptures
- AC joint separation

Contraindications

There are no known side effects. Please consult with your doctor if you suffer from any of the following:

- Diseases or injuries to the skin in the area in which the product is used, particularly if there are signs of inflammation (overheating, swelling or reddening)
- Changes in the sensitivity and perfusion of the lower arm and hand
- Changes in lymph drainage and swellings in the soft tissue away from the area of use and without a clear explanation
- Compression syndromes (pressure irritation to the nerves in the arms)

Bandages or orthopaedic devices can in principle cause local compression symptoms if they are applied too tightly. In rare cases, this may cause constriction of blood vessels or nerves.

Fitting instructions

The orthosis is normally fitted for the first time in the operating theatre. It should be put on and taken off on a

table wherever possible. Please pay attention to the following tips:
The snap fasteners on the front of the cushion should be opened. The shoulder strap already lies on the table.

1. Place the arm that has been operated on in the open pocket (Fig. 1).
2. Now close the pocket (Fig. 2).
3. Pass the waist strap around your back and close the snap fastener. (Fig. 3)
4. Guide the uninjured arm through the loop and pull the belt over the shoulder as you would with a rucksack. Lock the buckle clip. (Image 4+5)
5. In order to prevent the arm from slipping out, please attach the additional velcro fastening. (Image 6)

If you have the feeling during the night that the arm or shoulder is not securely positioned, you can put a pillow under the brace or you can tighten the strap around the waist.

In principle, this product must be used under medical supervision. When it is applied for the first time, this must be under the instructions of trained and specialised personnel, such as an orthopaedic technician. If the support is individually fitted by a qualified person, this will ensure that it is effective and comfortable.

If it is to be used with other products, this must be discussed with the prescribing doctor.

Care instructions

The product should not be worn in water. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Pillow case, bag and straps can be washed by hand, preferably using medi clean washing agent, or in the washing machine with the delicate cycle at 30°C using mild detergent without fabric softener.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

**Storage instructions**

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

PU foam, polyamide

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

Your medi team

wishes you a quick return to full fitness.

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