medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com medi

4 051526 816329

Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Orient M.E.A. ZCC
 TAURUS GROUP
 Suite No. M01 Wasel Al Mamzar Bldg.
 Al Wuhida Road, Al Mamzar
 P.O.BOX NO. 91464
 DUBAI, UAE
 United Arab Emirates
 T: +971- 4 2557344
 F: +971- 4 2557399
 md@taurusgroup.net

medi Polska Sp. z o.o.
 Zygmunt Starego 26
 44-100 Gliwice
 Poland
 T +48-32 230 60 21
 F +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
 Unit 1806, Floor 18, Sandhill Plaza No. 1
 2290 Zuchongzhi Road
 200131 Shanghai
 People's Republic of China
 T: +86-1366 1668 687
 F: +86-2161 7696 07
 info@medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
 Kültür mah. caddesi sok.14/3
 06420 Çankaya/Ankara
 Turkey
 T +90 312 435 20 26
 F +90 312 434 22 82
 info@medi-turk.com
 www.medi-turk.com

medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

E013300 / 02.2018

medi

medi SAS[®] light

Schulter-Außenrotations-Kissen External Rotation Shoulder Cushion Coussin d'abduction d'épaule Cojín para rotación externa del hombro

Gebruiksaanwijzing. Instructions for use. Mode d'emploi. Instrucciones de uso.
 Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Bruksanvisning.
 Návod k použití. Инструкция по использованию. Instrukcja zakładania. Használati
 útmutató. Navodila za uporabo. Návod na použitie. 使用说明. Instrucțiuni de utilizare.
 دليل الاستخدام.

medi. I feel better.

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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medi SAS® light

Intended purpose

The medi SAS light is an external rotation shoulder cushion. This product is only intended to be used for orthopaedic therapy on the shoulder and is only to be used on intact skin or when appropriate wound dressing is in place, and is only intended for patients.

Indications

- Conservative treatment of shoulder dislocation
- Postoperative treatment of
 - Rotator cuff tears
 - Shoulder instability
 - Shoulder fractures
- General shoulder injuries

Contra-indications

All indications that expressly require the shoulder to be held in an internally rotated position.

Putting on the cushion

- Position the wide strap beneath your chest and fasten it (Fig. 1).
- Position the external rotation cushion under your injured shoulder (Fig. 2) and fix it to your forearm with the velcro strap on the cushion. Your hand should sit comfortably on the hand rest. You can adjust the strap by cutting it to make it shorter if your forearm is narrower.
- Stick the shoulder strap that goes across your back to the strap that goes around your stomach (Fig. 4). Then bring the strap around your uninjured shoulder and close the clasp. Adjust the strap until load is relieved on the injured arm (Fig. 5).
- To prevent the arm slipping backwards, the additional strap should be fastened, particularly at night (Fig. 6)

Care instructions

Velcro fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear. The cover of the external rotation wedge should be removed before washing.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Polyamide, PU foam

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

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