medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T +32 011 24 25 60
 F +32 011 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Neusa 216
 Diadema-Sao Paulo
 Cep 09941-640
 Brazil
 T +55 11 3201 1188
 F +55 11 3201 1185
 sac@medibrasil.com
 www.medibrasil.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbæk Strand
 Denmark
 T +45 46 55 75 69
 F +45 70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

medi Bayreuth España SL
 C/Canigó 2 – 6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34 93 260 04 00
 F +34 93 260 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33 1 48 61 76 10
 F +33 1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1371 0090
 F +36 1371 0091
 info@medi.hu
 www.medi.hu

medi Italia S.r.l.
 Via Guido Rossa 28
 40033 Casalecchio di Reno-Bologna
 Italy
 T +39 51 613 24 84
 F +39 51 613 29 56
 info@medi-italia.it
 www.medi-italia.it

medi Medical Support Sdn Bhd
 medi representative office Asia
 Unit No. B-2-19, Block B, No.2,
 Jalan PJU 1A/7A
 Oasis Ara Damansara, PJU 1A,
 47301 PETALING JAYA
 Selangor Darul Ehsan
 Malaysia
 T: +6 03 7832 3591
 F: +6 03 7832 3921
 info@medi-asia.com
 www.medi-asia.com

medi Middle East
 P. O. Box: 109307
 Abu Dhabi
 United Arab Emirates
 T +971 2 6429201
 F +971 2 6429070
 info@mediuae.ae
 www.mediuae.ae

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31 76 57 22 555
 F +31 76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o. o.
 Zygmunt Starego 26
 44-100 Gliwice
 Poland
 T +48 32 230 60 21
 F +48 32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua Eugénio de Castro Rodrigues, nº2
 1700-184 Lisboa
 Portugal
 T +351 21 843 71 60
 F +351 21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova st., 17
 117342 Moscow
 Russia
 T +7 (495) 374-04-56
 F +7 (495) 374-04-56
 info@medirus.ru
 www.medirus.ru

medi Sweden AB
 Box 6034
 192 06 Sollentuna
 Sweden
 T +46 8 96 97 98
 F +46 8 626 68 70
 info@medi.se
 www.medi.se

medi UK Ltd.
 Plough Lane
 Hereford HR4 OEL
 United Kingdom
 T +44 1432 37 35 00
 F +44 1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

medi Ukraine LLC
 Tankova Str. 8, office 35
 Business-center "Flora Park"
 Kiev 04112
 Ukraine
 T +380 44 591 1163
 F +380 44 455 6181
 office@medi.ua
 www.medi.ua

medi SAS® comfort

Schulterabduktionsschiene Shoulder abduction splint Orthèse d'abduction de l'épaule Férula para abducción de hombro

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Istrucciones de uso.
 Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning.
 Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкция по использованию.
 Kullanma kılavuzu. Instrukcja zakładania. Οδηγία εφαρμογής. Használati útmutató.

medi

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Important notes

This product is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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medi SAS[®] comfort

Intended Purpose

medi SAS comfort is a shoulder support cushion. The product is exclusively to be used for the orthotic fitting of the shoulder and only on unbroken skin.

Indications

- Postoperative immobilisation following the implantation of shoulder prostheses
- Immobilisation after the reconstruction of rotator cuff ruptures
- AC joint separation

Contraindications

There are no known side effects. Please consult with your doctor if you suffer from any of the following:

- Diseases or injuries to the skin in the area in which the product is used, particularly if there are signs of inflammation (overheating, swelling or reddening)
- Changes in the sensitivity and perfusion of the lower arm and hand
- Changes in lymph drainage and swellings in the soft tissue away from the area of use and without a clear explanation
- Compression syndromes (pressure irritation to the nerves in the arms)

Bandages or orthopaedic devices can in principle cause local compression symptoms if they are applied too tightly. In rare cases, this may cause constriction of blood vessels or nerves.

Fitting instructions

The orthosis is normally fitted for the first time in the operating theatre. It should be put on and taken off on a table wherever possible. Please pay

attention to the following tips:

The snap fasteners on the front of the cushion should be opened. The shoulder strap already lies on the table.

1. Place the arm that has been operated on in the open pocket (Fig. 1).
2. Now close the pocket (Fig. 2).
3. Pass the waist strap around your back and close the snap fastener. (Fig. 3)
4. Guide the uninjured arm through the loop and pull the belt over the shoulder as you would with a rucksack. Lock the buckle clip. (Image 4+5)
5. In order to prevent the arm from slipping out, please attach the additional velcro fastening. (Image 6)

If you have the feeling during the night that the arm or shoulder is not securely positioned, you can put a pillow under the brace or you can tighten the strap around the waist.

In principle, this product must be used under medical supervision. When it is applied for the first time, this must be under the instructions of trained and specialised personnel, such as an orthopaedic technician. If the support is individually fitted by a qualified person, this will ensure that it is effective and comfortable.

If it is to be used with other products, this must be discussed with the prescribing doctor.

Care instructions

The product should not be worn in water. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Pillow case, bag and straps can be washed by hand, preferably using medi clean washing agent, or in the washing machine with the delicate cycle at 30°C using mild detergent without fabric softener.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Consistent with legal guidelines.

Disposal

The product can be disposed of in the domestic waste.

Your medi team

wishes you a quick return to full fitness.

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